i-See Touch

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Read this manual carefully before using the I-See Touch.
Keep the user manual for future reference.

Table of contents

EN

1	Foreword	17
2	Safety	17
3	Device overview	18
	Contents of the box	18
	Controls	18
	Connections.....	19
	Installation.....	19
4	Use	20
	Turning the device on or off	20
5	Live view mode	20
	Controls	20
	External input.....	21
6	Live view settings	22
7	Troubleshoot	24
	Maintenance	25
8	Recycling.....	25
9	Warranty	25
10	Technical specifications	26
11	Remote control	52
12	Supplier information	54
13	Manufacturer information.....	54

This document concerns the original general user manual,
has been prepared by and is commissioned by Rehan Electronics Ltd.
The I-See, I-See Touch product names are trademarks of Rehan Electronics Ltd.

1 Foreword

This user manual is intended to describe this device's use, installation and maintenance. Read this manual carefully before use and observe all safety instructions to prevent damage to your health, the device or the environment.

The I-See Touch is designed to support reading for people with low vision or dyslexia. The device is intended to enlarge documents/media up to 65x, and enhance contrast. This is possible for documents on paper.

The device is intended as a desktop for in-home or office use, with easy-to-use controls on the display panel and optional remote control. The large A3-size tableau makes it easy to move the reading line back and forth.

In addition, you can connect external sources by HDMI to display on the device.

EN

2 Safety

- Place the I-See on a stable and horizontal surface to prevent it from falling.
- Ensure that the I-See is placed at least 10 cm / 4 inches from the edge of the table.
- Only use your I-See indoors. Do not expose the device to extreme temperature fluctuations or humidity. Do not place your I-See next to a heat source.
- Ensure that the I-See is placed in a well-ventilated room.
- Only connect the adaptor to a wall socket outlet using the supplied cables.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the I-See, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Use precautions for implanted devices, like a pacemaker. Maintain at least 15cm/6 Inches between the speaker or headphone and the implanted device. The device also contains magnets in the display's upper corners and in the tableau's remote holder. See magnetic strength details in the technical information. If in doubt, contact your specialist.
- Be careful when handling the moving parts of the tableau of the screen.
- Do not use the I-See or its tableau to support walking, sitting down or standing up.
- Switch off the device if you do not intend to use it.
- The device may give possible reading errors due to external influences such as ambient light, paper type or print quality. Check critical data such as doctor prescriptions or bank details with other persons.

3 Device Overview


Contents of the box

- I-See Touch device
- Power Adaptor
- User Manual
- I-See training Doming Label
- HDMI Cable (optional for models with switch functionality)
- Remote Control and Remote control receiver (optional accessory)

Controls

The device has separate touch controls underneath the screen.

EN

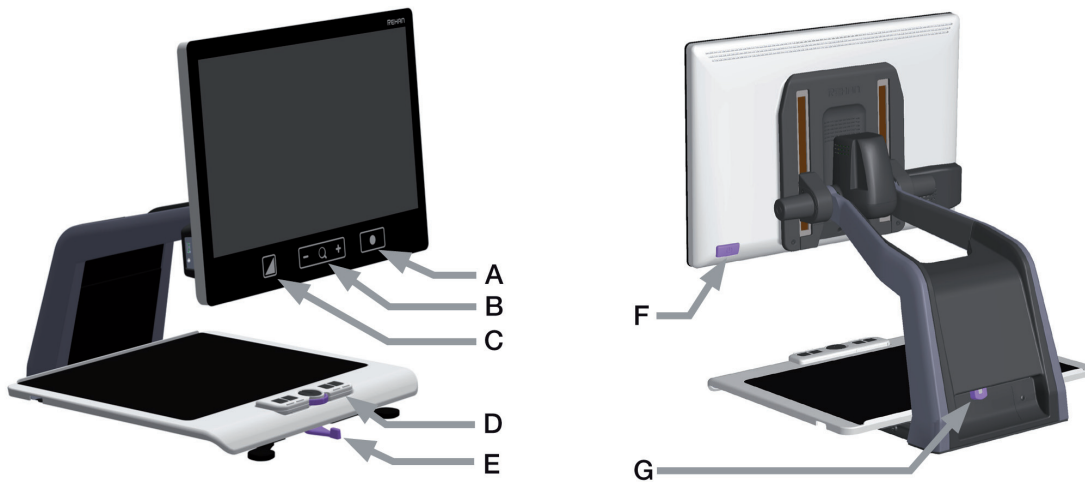- A - Touch button 3 (B3)
- B - Touch button 2 (B2)
- C - Touch button 1 (B1)
- D - Remote Control

- E - Brake handle
- F - ON/OFF button
- G - Connections

The optional remote control allows you to control the device without the touch buttons below the screen.

Depending on which model you have, it has 2 Touch functions or multiple buttons for the advanced functions.

See chapter 11 for more information.

Tableau


The I-See Touch contains a tableau on which you can place your reading material. The tableau can move from left to right and from front to back.

Brake handle

In front of the tableau of the I-See Touch, you will find a single brake handle with which you can control the movement of the tableau.

The handle has four positions:


- Left: The tableau can be moved freely.
- Middle-left: Free movement from left to right. The the front-to-back di-
rection can be adjusted in
5-millimetre increments.
- Middle-right: Free movement from left to
right. Front-to-back direction is
blocked.
- Right: Tableau position is completely
locked.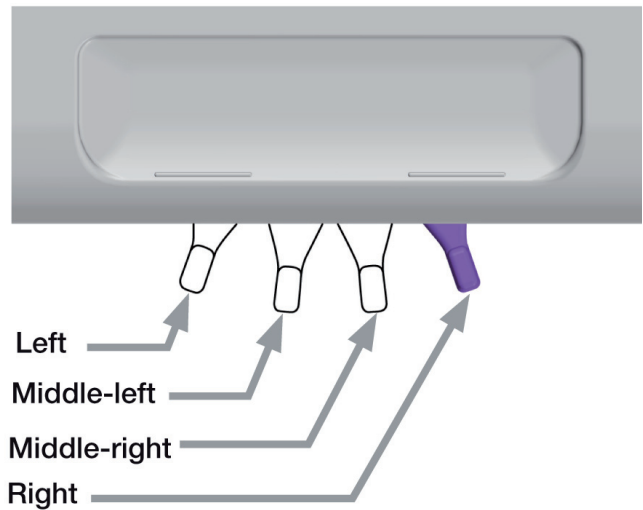

Connections

At the back of the device:

- Power
- Remote control receiver connection
- 2x HDMI (for models with switch func-
tionality)

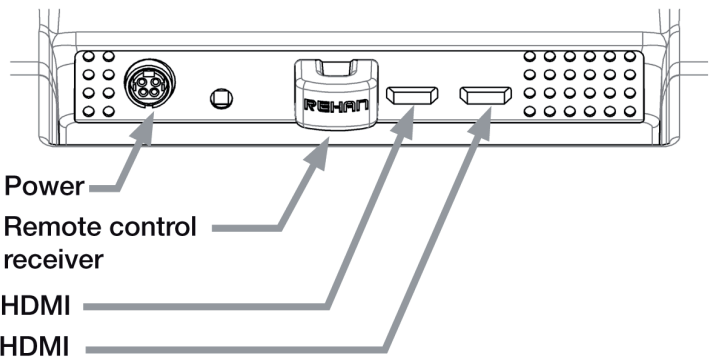Installation

The I-See will be delivered to you in a pre-assembled state.

Get the device out of the box

The top of the box can be opened, allowing the device to be lifted out of the box. Note that the weight of the device is around 17 kg. Lift the device from the back by holding both arms. Place the I-See on a stable and horizontal surface. Remove the foam blocks and store them in the original box.


Positioning

Height:

The height of the screen can be adjusted by firmly holding the screen on the left and right side, and pulling the screen up or down to your preferred position.

Tilt:

The screen can be tilted by firmly holding the screen on the left and right side and tilting the screen in your preferred position.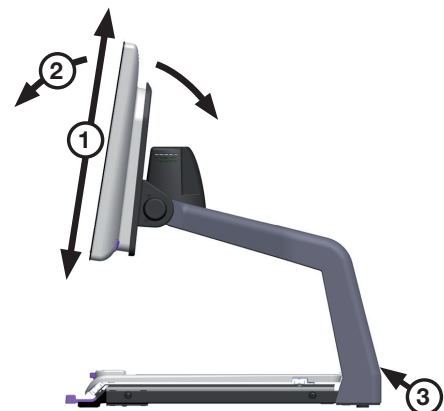

Power adapter

Place the round connector from the power adapter to the power input at the back of the device. It will fit only

in one direction so keep the flat part of the connector towards you. Connect the other end of the power adapter into the wall socket.

4 Use

Turning the device on or off

The on/off switch of the I-See Touch is located at the lower right hand side on the back of the monitor (see the Controls pictures in chapter 3). To start up or shut down the device, hold it for 2 seconds until you hear the startup or shutdown tone.


During start up the logo appears on the screen and when this disappears, the live image of the camera is displayed. This is the “Live view mode” of the device. The device is now ready to use.

5 Live view mode

When you turn on the device it will start in Live view mode. In the Live view you will see the tableau or the documents on it, which you can use for enlargement or contrast improvement.

Controls

The three buttons (B1, B2 and B3) can be found below the screen and consist of touch sensitive pads.

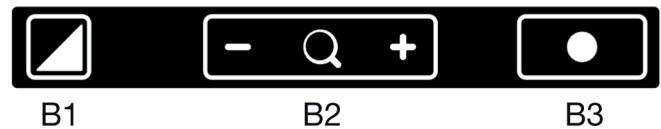For improved feel of the position of these touch sensitive pads, the I-See Training Doming Label can be pasted over these touch sensitive pads.

The functions of the three touch sensitive pads are as follows:

Display modes

Tap this icon (B1) to go to the next available display mode.

Press and hold this icon to quickly switch between your last used high contrast mode and photo mode.

Swipe over this icon from left to right or right to left to go to the next or previous available display mode. “Photo mode” is the icon with multiple colors (A).

Zoom

Tap or press and hold the minus or plus icon to zoom out or in (B2).

The magnification level will be shown on the screen.

Block auto focus

Tap the magnifier icon to refocus on the reading material and block the auto focus on that level. Tap a zoom button to unblock the auto focus.

Brightness

By tapping this icon (B3) you can cycle through 5 brightness levels.

Brightness can't be adjusted when reading aids are in use.

B1


B2

B3

Contrast

In contrast mode, swipe over this icon (B3) from left to right (and hold your finger on the right side of the rectangle) or right to left (and hold your finger on the left side of the rectangle) to gradually increase or decrease the contrast. The contrast icon is shown in grayscale.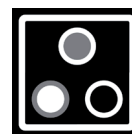

Contrast can't be adjusted when reading aids are in use and can only be adjusted in high contrast modes, not in photo modes.

White balance

In photo mode, swipe over this icon (B3) from left to right (and hold your finger on the right side of the rounded rectangle) or right to left (and hold your finger on the left side of the rounded rectangle) to gradually change the white balance to the colder, more blue side or change the white balance to the warmer, more orange side. The white balance icon has multiple colors.

White balance can't be adjusted when reading aids are in use and can only be adjusted in photo modes, not in high contrast modes.

White balance adjustment is turned off by default. To turn it on read Chapter 6 Live view settings.

Advanced reading aids


Press and hold the brightness icon (B3) to enter the Advanced reading aids menu. The following reading aids are available:


horizontal
blanking

vertical
blanking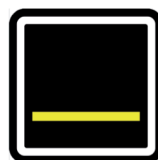

horizontal line

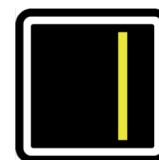vertical line

Swipe up or down over this icon to select the desired reading aid required. Each reading aid has 5 settings that are indicated by a number shown in the icon.

Once the desired reading aid is selected, tap this icon to set the desired setting of the reading aid. To change the setting value you can tap this icon to go to the next level or you can swipe from left to right.

To change to a different reading aid then swipe up or down over this icon and then tap to set the desired level of the reading aid selected.

To exit from the advanced reading aids menu press and hold this icon (B3) for 2 seconds. The reading aid will now disappear from the screen.

Brightness, contrast and white balance can't be adjusted when reading aids are in use.

External input

The I-See with the switch functionality has the possibility to connect an external device, such as camera or computer, by HDMI to the I-See. The image from this external device will be displayed on the screen of the I-See.

Press the ON/OFF button once while the device is on, to switch between the internal camera and one of the external inputs. Once a remote control is connected, you can press the external input button on the remote control. Switching to an external input

is only available when an HDMI cable is connected to one of the HDMI ports at the back of the I-See and to an external device (see the Connections picture, chapter 3.).

On the image of the external input, it is possible to zoom and adjust display modes, brightness, contrast, white balance and advanced reading aides. These functions work in the same way as explained earlier.

Tap or press and hold the minus or plus icon to zoom out or in. When zoomed in, it is possible to pan through the image by pressing and holding the magnifier icon and sliding your finger in the direction you want to pan the image. Always start from the magnifier icon to start panning.

6 Live view settings

EN




The I-See Touch contains a settings menu with which you can (de)activate certain functions of the Live view mode. You can access the settings menu by ensuring the device is switched off, then turning the device on using the on/off button; press and hold until you hear a beep after the start up tone.

By tapping the brightness icon (B3) you can cycle through multiple screens of selectable options. Tap the plus icon of the zoom button (B2) to highlight which mode you wish to change.

You can recognize the highlighted function by the white rounded rectangle around it. Tap the magnifier icon to activate/deactivate your selection.

The alternative method is by using the remote control. The wheel is used for highlighting the function. The button on the right ('external input') to cycle through multiple screens. With the button on the left you can activate/deactivate your selection.


The process of saving the changes and exiting is described at the end of this chapter.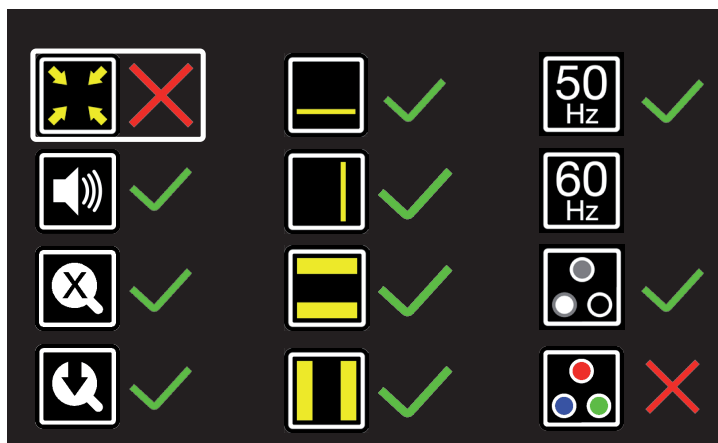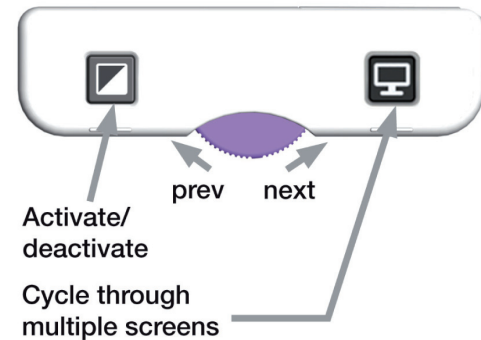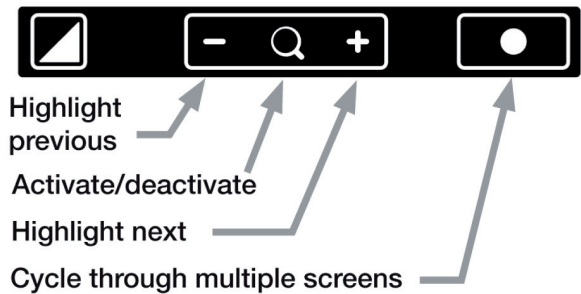

Display modes

On the first screen you can select whether you want the photo mode to have realistic or high saturation colors by activating the photo mode (A) and select which high contrast mode you want available in your Live view mode by activating the high contrast mode you want activated/deactivated.

The second screen shows advanced functions:

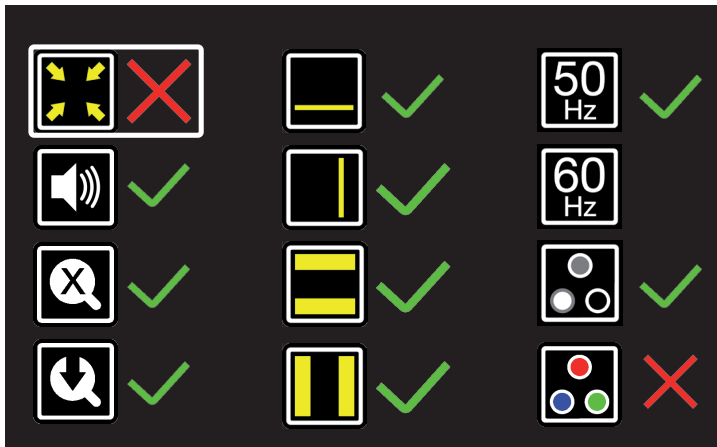by default.

Crop zoom

The crop zoom function crops the image on screen to 50% so that a zoom level of 1x can be achieved. *This function is turned off by default.*

Sound feedback

When this function is on you hear a beep when you change a function. *The function is turned on*

Magnification indicator

When you zoom in or out a magnification indicator shows up in the top left corner that shows what the magnification factor is. *This function is turned on by default.*

Zoom preset

When this function is on the device will go to the last used magnification in high contrast modes and photo mode when you switch to these functions. *This function is turned on by default.*

Advanced reading aids

This will let you turn reading aids on or off individually. *This function is turned on by default.*


Refresh rate / frequency

Depending on your location the mains power frequency will be 50 or 60 Hertz. By selecting the right setting you will suppress flickering caused by surrounding light. *This function is set to 50Hz default.*


Contrast

With this function you can turn the manual contrast adjustment on or off. *This function is turned on by default.*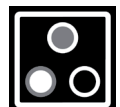

White balance

Select whether you want the ability to manually set the white balance in the photo modes. *This function is turned off by default.*

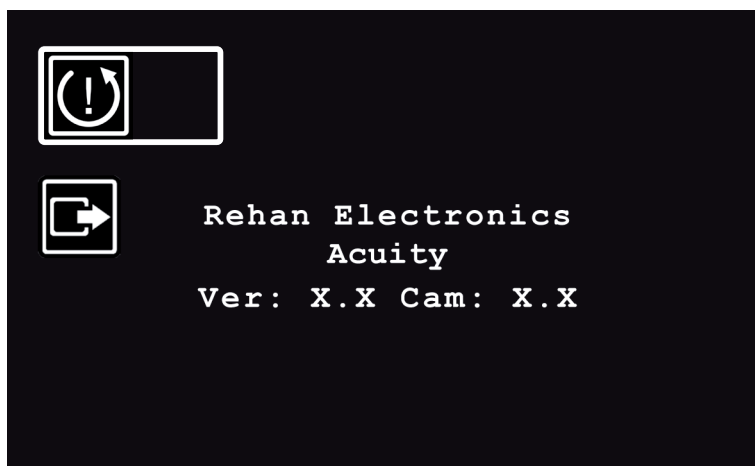On the third and last page of the settings you will find the factory reset, save and exit and the installed software version.

Factory reset

Activating this function and turning the device off and on again will reset all advanced functions and last used magnifications to their default setting. *The factory reset function is a*

permanent action and can not be undone. This function is turned on by default.

Save and exit

Activating this function saves any changes and exits the menu. If you didn't make changes, the device stays on and can be used immediately. In case you made changes, the device will automatically go through a power cycle (turn off and on again) and the changes are saved.

7 Troubleshoot

General

Problem description	Actions
<p>The remote control is not responding.</p> <p><i>Please note: It can take around 10 seconds to create the wireless connection after pressing a button.</i></p>	<p>Is the remote control receiver (dongle) connected to the I-See at the back of the device?</p> <p>Is the battery empty of the remote control? It uses a single AAA battery.</p>
<p>The device does not switch to external input.</p>	<p>The I-See can serve as a monitor for a computer. Connect the HDMI cable between the PC and one of the two HDMI inputs at the back of the device.</p> <p>The recommended resolution should be 1920x1080.</p> <p>Press the power button until the source gets available or press the external input button on the remote control.</p> <p>If an X is displayed, check if the cable is well inserted into the ports.</p>

Live view mode

Problem description	Actions
<p>The image or text is blurry and not in focus.</p>	<p>Tap the magnifier icon (B2) to refocus. The focus is now blocked. Zoom in or out to activate the autofocus again.</p>
<p>The text is not good in high-contrast mode.</p>	<p>The contrast can be adjusted by swiping over the button (B3) from left to right, or right to left and holding your finger pressed until the letters become thicker.</p> <p>Make sure the function is activated in the "Live view settings".</p>
<p>In the photo, the image is yellowish or blueish.</p>	<p>The white balance can be adjusted by swiping over the button (B3) from left to right or right to left and holding your finger pressed until the color gets more natural.</p> <p>Make sure the function is activated in the "Live view settings".</p>
<p>Part of the image is blanked.</p>	<p>The blanking reading aid is on. Press and hold the button (B3). Page blanking is switched off.</p>

Maintenance

Cleaning


For safety reasons, always disconnect the AC power adapter from the mains supply before cleaning the I-See.

Follow the instructions below to help prevent the product exterior from deteriorating or discolouration:

- Wipe with a soft, dry cloth.
- Do not use solvents or other chemicals. Do not wipe with a chemically-treated cleaning cloth.
- Use a soft dry cloth to clean all other accessories.

8 Recycling

The I-See Touch contains electrical and electronic equipment (WEEE) and should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will usually be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

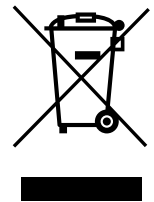Disposing of this product and the packaging correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. If you are unsure, contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

9 Warranty

This product is covered by the warranty provisions of your supplier. For more information about these provisions, contact your supplier. Never hesitate to contact your supplier or the manufacturer to report unexpected operation of incidents. Please note the serial number of the device when contacting the manufacturer.

Manufacturer address — Rehan Electronics Ltd.,
production date — 2021-05

Catalog reference number — **REF 2000-21-0015-22B**

UDI number — **UDI** (01)05391513470707
(21)AC00000

Serial number — **SN AC00000**

Device input power — **DC in: 12V, 5A**

WEEE recycling —

Medical device purpose and name — **MD Electronic desktop magnifier**

Read the instructions for use —

Complies with Part 15 of the FCC Rules —

Product conform E.U. standards — **FC CE**

Indoor use temperature — **IP20 40°C 5°C**

IP rating —

10 Technical specifications

Resolution		1920 × 1080	
Video refresh rate		50 Hz or 60 Hz	
Display size		54.6 cm 22 inch	
Display modes <i>These display modes can be turned off in the I-See live mode advanced menu</i>	Photograph	Full colour (or high saturation)	
		Greyscale	
	High contrast	Black - white	Black - yellow
		White - black	Yellow - blue
		Green - black	Blue - yellow
		Black - green	Red - black
Yellow - black		Black - red	
Magnification (With a 22" screen)	Minimum	≈ 2.3x (when in crop mode ≈1x)	
	Maximum	≈ 65x	
Reading Aid	Horizontal Reading Line	Available in all models	
	Vertical Reading Line	Available in all models	
	Horizontal Block Blank	Available in all models	
	Vertical Block Blank	Available in all models	
Preferred external HDMI input setting		1920x1080	

If you have any questions after reading this manual, please contact your supplier.

Power Supply		Model: CGSW65-120-5000II Input: 100-240VAC, 50/60Hz, 1.5A max. Output: 12VDC, 5000mA max.
Power rating		Standby: 0.2W Maximum: 60W
Dimensions	22" (screen in highest position)	48 × 48 × 65 cm 18.9 × 18.9 × 25.6 inch
Weight		17 kg
Magnets		Upper corners of display: <i><0.5mT at 7cm distance</i> Remote control holder at tableau: <i><0.5mT at 4cm distance</i>
IP rating		IP20; <i>solid foreign objects ≥ 12.5mm</i> <i>Not protected against ingress of water</i>
Conditions of use	Temperature	5 to 40 °C 41 to 104 °F
	Humidity	0 to 75%, no condensation
Conditions of storage and transport	Temperature	-25 to 70 °C -13 to 158 °F
	Humidity	0 to 90%, no condensation

FCC NOTICE (Class B)

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. this device may not cause harmful interference, and
2. this device must accept any interference received, including interference that may cause undesired operation. Federal Communications Commission Statement.

EMI:

The I-See Touch complies to harmonised standards according to medical devices and with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications.

However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/television technician for help.

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